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Description:

The Original CLEARsplint® Disc contains a cured disc of CLEARsplint® to be used in a CAD/CAM machine to produce an occlusal splint for suffering patients.

Intended use: The Original CLEARsplint® Disc is intended to be used by a dentist in the laboratories to fabricate flexible occlusal splints for patients with tooth grinding, TMJ, bruxism and for interims. The Original CLEARsplint® Disc is to be worn daily, nightly or for continuous use. Interims can be used continuously for up to six months in patients after an extraction to allow the gum and bone to heal prior to an implant restoration.

Mechanism of action: Polymerized disc used in a CAD/CAM machine to create a dental appliance.

Operating/handling instructions: Avoid prolonged breathing of vapors while milling and use with adequate ventilation. Do not smoke in the vicinity of the disc. Wear protective gloves and safety goggles with side shields. See Safety Data Sheet (SDS) for complete information which is available at astrondental.com or by contacting the manufacturer.

Warning: Combustible

Handling: For removing the milled splint or interims out of the disc, standard type burs or separating discs as suited for resins should be used. Polishing with pumice or with polishing paste and buffing wheels as usual.

Caution: An allergy test is recommended prior to placing in an oral cavity.

Warranty: Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used as instructed or not stored in its original container.

Disposal: Dispose of material in accordance with local, state and federal laws.

Storage conditions: Store in original package in a cool, dry, well-ventilated area. Keep away from sources of ignition and light.

Other materials/equipment required: CAD/CAM Milling Machine

Supply: This IFU is applicable for the following packages

26531	The Original CLEARsplint® Disc 1	Unit 95x20* with Clover Edge
26541	The Original CLEARsplint® Disc 3	Unit 95x20* with Clover Edge
26551	The Original CLEARsplint® Disc 12	Unit 95x20* with Clover Edge
26533	The Original CLEARsplint® Disc 1	Unit 98.5x20* with Shoulder
26543	The Original CLEARsplint® Disc 3	Unit 98.5x20* with Shoulder
26553	The Original CLEARsplint® Disc 12	Unit 98.5x20* with Shoulder
26532	The Original CLEARsplint® Disc 1	Unit 98x20* without Shoulder
26542	The Original CLEARsplintv Disc 3	Unit 98x20* without Shoulder
26552	The Original CLEARsplint® Disc 12	Unit 98x20* without Shoulder

Instructions: Place the correct size Original CLEARsplint® Disc in the CAD/CAM machine. Follow the instructions for your specific CAD/CAM machine to correctly mill the disc. If dry milling, reduce the burr speed by approximately 20% to prevent overheating. Wet milling will produce optimum results. When using the wet milling method, the surface will be more shiny. Polish and trim as usual. Immerse in 50°C/122°F warm water for twenty seconds prior to placing on mold or in the mouth to prevent breakage.

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Kentzler-Kaschner Dental GmbH